


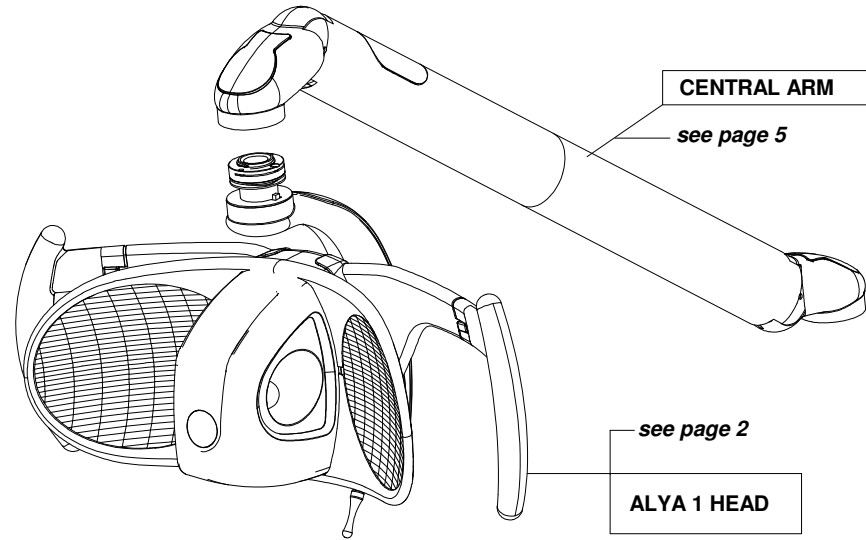



CABLES *page: 8*



see page 7


**REAR ARM
WITHOUT TRANSFORMER**

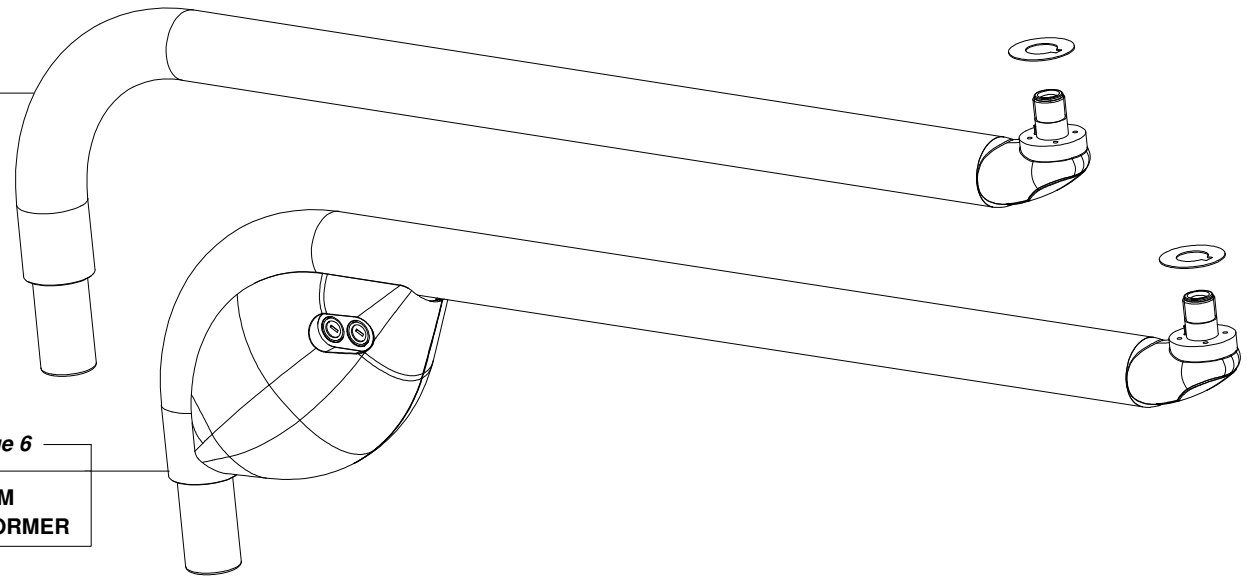

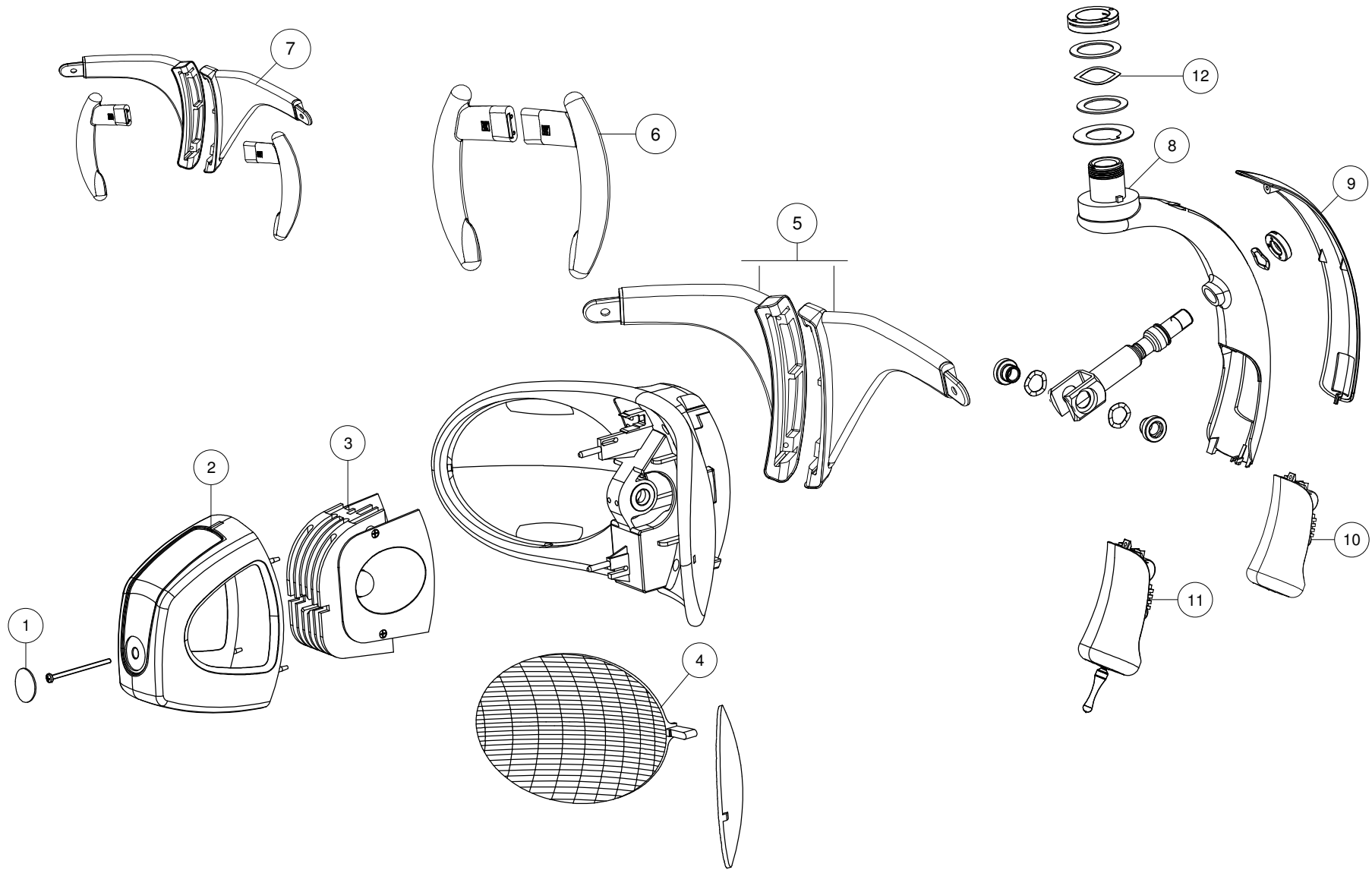





Pos	Code	Description
1	SP408015	Lable set
2	SP310017	Led assembly cover grey
3	SP310200	Optical group 5000°K Alya 1st series
4	SP310025	Alya reflectors
5	SP310352	Grey handle holder set rv2
5	SP310352006	White handle holder set rv2
6	SP310370	Grey handle set rv2
6	SP310370006	White handle set rv2
7	SP310390	Complete handle set rv 2 Gray
7	SP310390006	Complete handle set rv2 White
8	SP310027	Complete junction joke
9	SP310036	Gray cover set
10	SP310130	Proximity switch with gray cover for Alya first edition
12	SP310045	Head fixing set

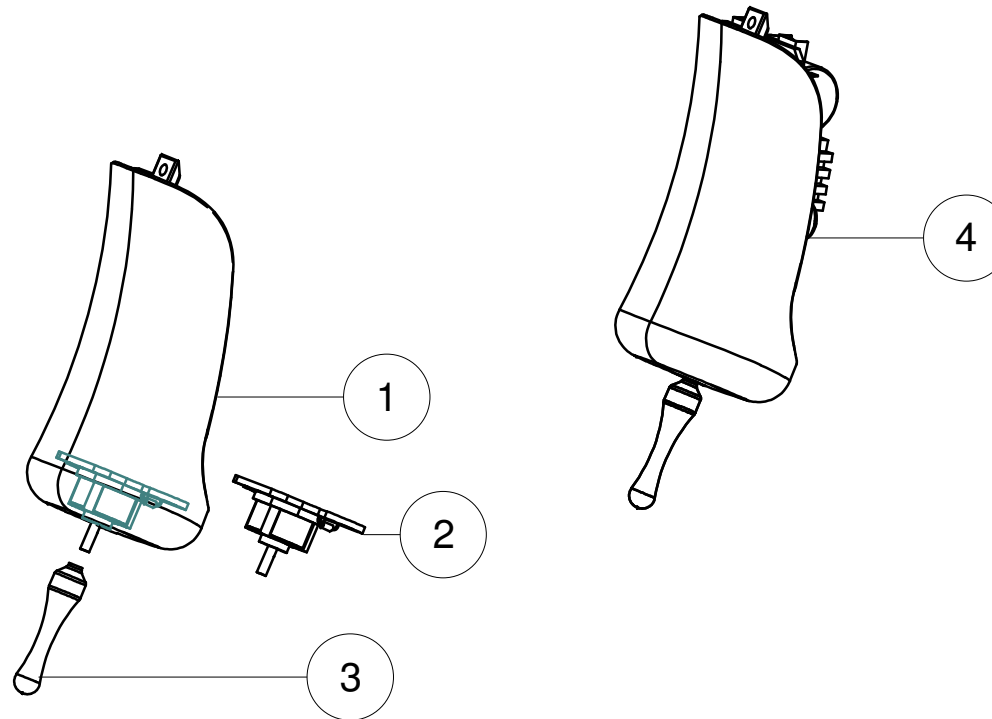


Pos	Code	Description
1	SP310123	Joystick with gray cover
2	SP310102	Joystick Alya
3	SP310108	Lever for joystick switch
4	SP310125	Complete gray Alya switch first version

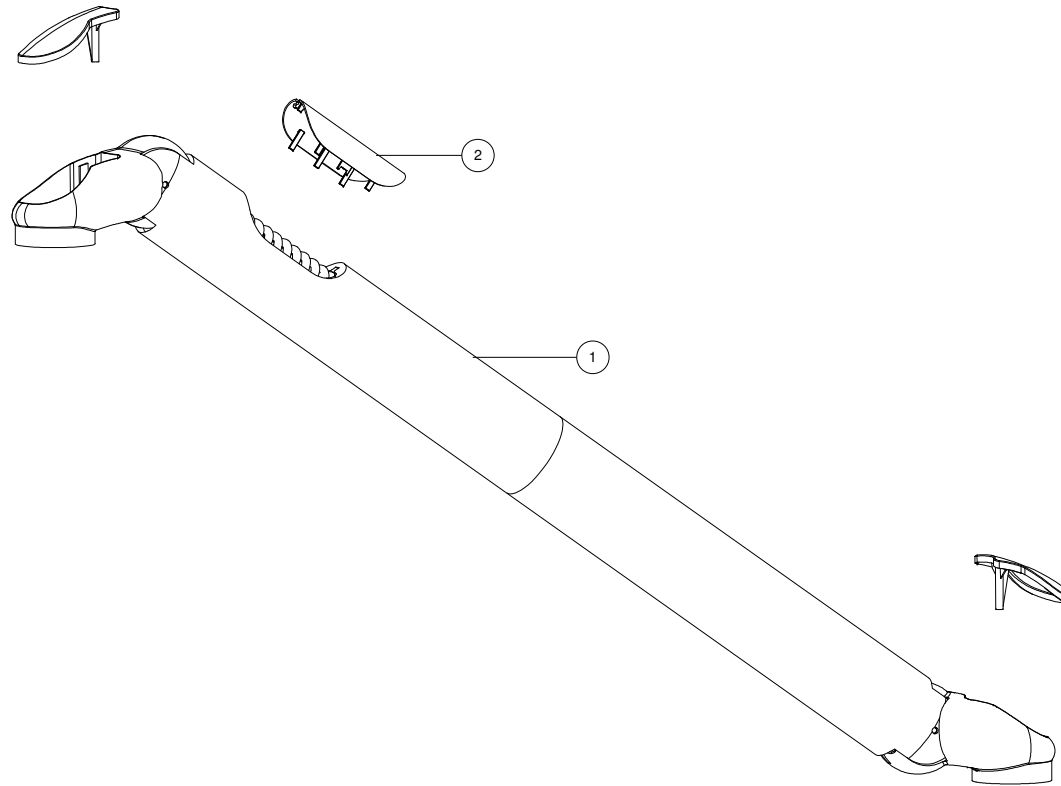

Pos	Code	Description
2	SP310036	Gray cover set
1	SP310451	Central arm lenght 550mm for Alya with transformer
1	SP310453	Central arm lenght 855mm for Alya with transformer
1	SP310450	Central arm lenght 550mm for Alya without transformer
1	SP311450	Central arm lenght 550mm for Alya without transformer ceiling version
1	SP310452	Central arm lenght 855mm for Alya without transformer
1	SP311452	Central arm lenght 855mm for Alya without transformer ceiling version

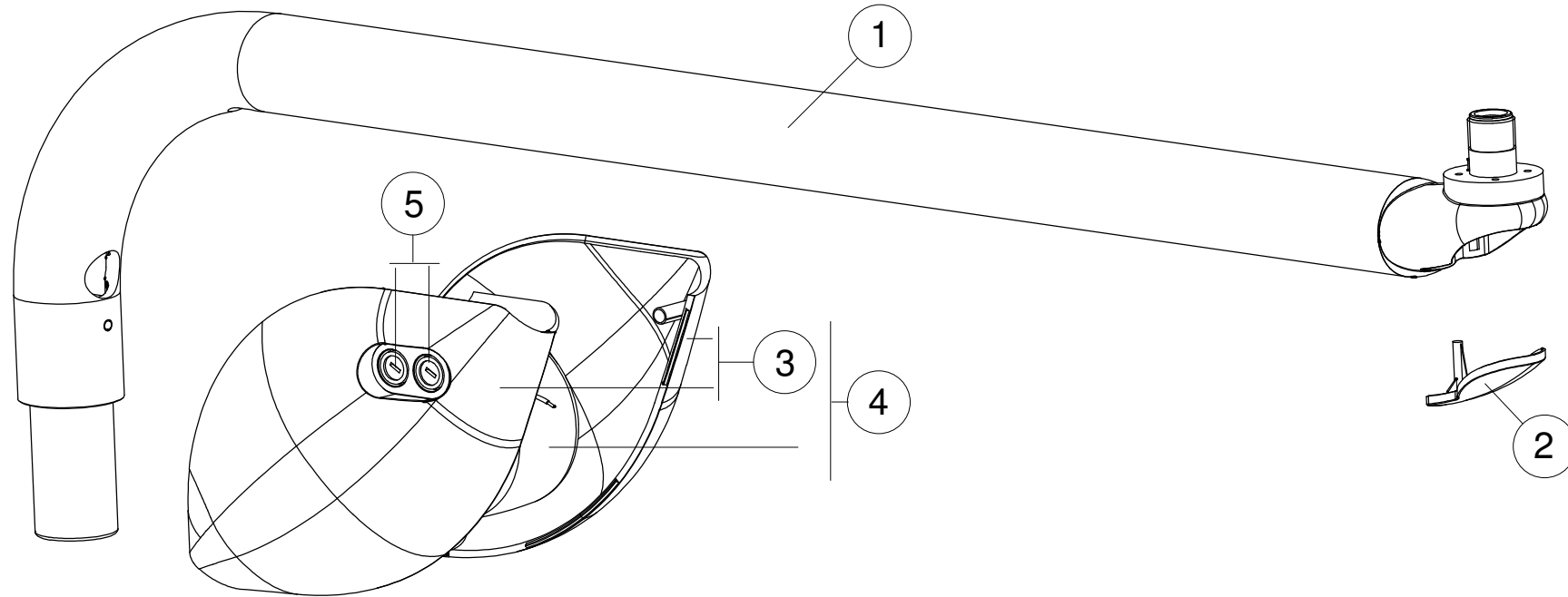

Pos	Code	Description
1	SP310157	Rear arm lenght 810mm for Alya with transformer
1	SP311401	Rear arm lenght 810mm for Alya with transformer
1	SP310411	Rear arm lenght 960mm for Alya with transformer
1	SP311411	Rear arm lenght 960mm for Alya with transformer
2	SP310036	Gray cover set
3	SP306155	Edi and Alya transformer cover
4	SP310405	Alya 230V transformer
5	SP310406	Alya 230V fuse set

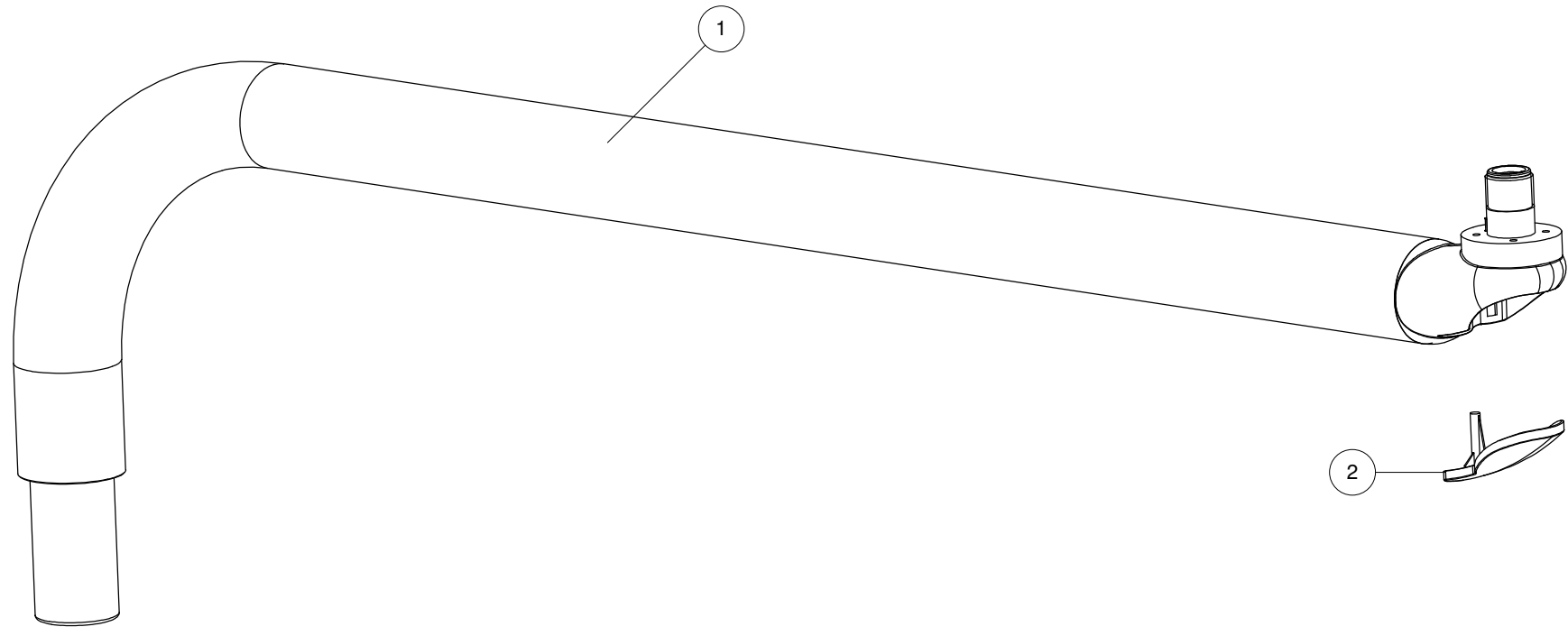

Pos	Code	Description
1	SP310150	Rear arm lenght 810mm for Alya without transformer
1	SP311150	Rear arm lenght 810mm for ceiling Alya without transformer
1	SP310155	Rear arm lenght 960mm for Alya without transformer
1	SP311155	Rear arm lenght 960mm for ceiling Alya without transformer
2	SP310036	Gray cover set



Code	Description
SP310095	Cable from lamp setto pcb
SP310139	Cable from pcb to lamp set
SP310138	cable from pcb to power cable
SP310104	Alya horn
SP310146	Remote control cable
SP310075	Alya Ts Power cable
SP311075	Alya Ts Power cable ceiling version
SP310413	Power cable for lamp S Type