

17 If the Led "Open Cover" (look at image bridge sector C pos 31) does not turn off when the cover is closed

- Verify the running of the micro SB3. Look at picture 30.
- Check the cam pos. C Picture 30 Micro functions operation.
- In order to regulate the distance of the cam from the micro unscrew the screw pos. A, acting also on the lock nut pos. B picture 30.

18 If the Led "Open Door" (look at image bridge sector C pos 32) does not turn off when the cover is closed

- Check the two controls in series which direct this led.
 1. Check that the micro-switch in rest position on the door's shutting is NC (normally closed). Look at picture 31.
 - a) The micro-switch is ruled by the horizontal movement of the plate pos. B. (Look at picture 31)
 - b) Check that the micro-switch when pressed changes over circuit NA (normally open)
 - c) In case the micro-switch doesn't commute, replace it.
 2. The second control which directs the led arrives from CAST button. (Look at bridge picture pos. 32)
 - a) Check the correct running of the button in pos. A with the tester. (Look at picture 32)
 - b) Check the button's welding in pos. A. (Look at picture 32)
 - c) Eventually replace it